




Introducing
a novel design!

“
Mandibular Advancement Device
For Snoring & Sleep Apnea Treatment.
”

Kyung Hee Marronnier Orthodontic Dental Lab





Address Store No.204
21, Im un-ro 1-gil, Dongdaemun-gu, Seoul,
South Korea 130-792
Tel 82-2-969-2322
Fax 82-2-969-2352
E-mail wellex28@naver.com
Blog bolg.naver.com/brmentor
Mobile 82-10-3255-5365

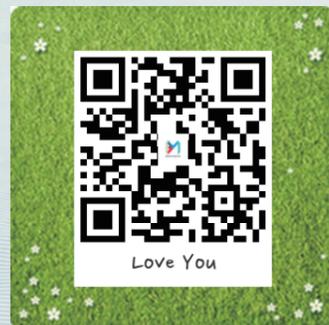



Features of “Breath Mentor”

- 1 / Intraoral adjustment and placement are made easy with separate maxillary and mandibular pieces.
- 2 / The screw located at the maxillary anterior region allows an easy adjustment of mandibular advancement.
- 3 / A slight advancement of the mandible allows to open mouth.
- 4 / Minimal wear and high tolerance to lateral (excursive) movements due to its simple structure.
- 5 / Helps to minimize the occurrence of an ulcer and damages to the buccal mucosa by alleviating compression of the cheek when sleeping sideways.
- 6 / Designed to allow slight lateral (excursive) movements of bruxism which may occur during REM sleep.
- 7 / **Adjustment for extra vertical range of motion**
For patients who need mouth breathing or an extra vertical dimension, please remove with a bur the resin saw-tooth shown in red in the diagram on the left.

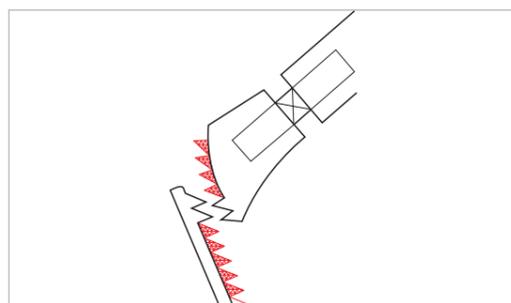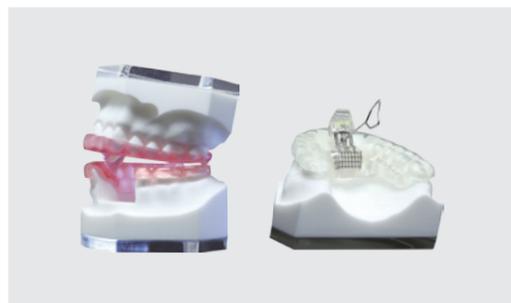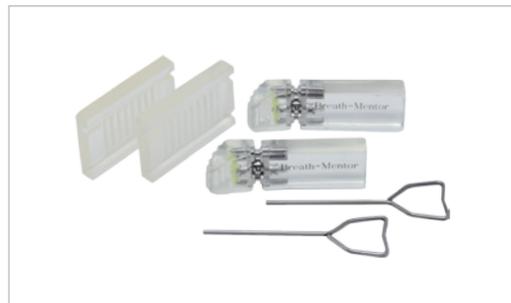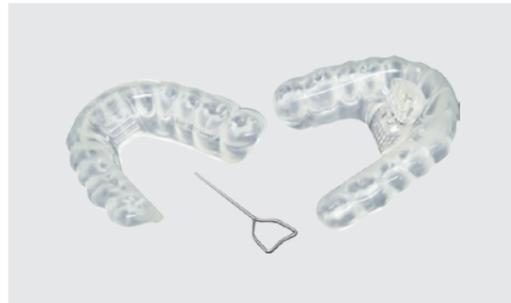

User's Guide

- 1 / Breath-mentor is an innovative intraoral device that helps to open your airway with easy by advancing your lower jaw forward. It is effective in the prevention of snoring and obstructive sleep apnea.
- 2 / The mandibular piece is originally set 4mm advanced in respect to the centric relation of the user.
- 3 / Each user may have a different degree of the mandibular advancement when using the device. In general, the mandibular piece is advanced 6~7 mm. However the degree of the advancement can be adjusted based on the user's discomfort level as well as the degree of snoring.
- 4 / Advancement of the mandibular piece can be achieved by placing the key on the metal key hole located in the maxillary piece. Rotate the key following the arrow for one click in order to expand 0.2mm. 5 clicks advance the mandibular piece by 1mm.
- 5 / The mandibular piece comes originally 4mm advanced in respect to the centric relation. The device can be used with additional 2~3 mm expansion.

For further information, please contact us.

Contact information is provided on the next page. Thank you.

We are looking for partners

There is an increasing number of people who suffer from snoring and obstructive sleep apnea, and interests in the therapy for snoring and OSA are growing.

The market for intraoral devices such as Mandibular Advancement Device is exponentially growing as experts in the field agree on its effectiveness and convenient use.

At this exhibition we are looking for a partner from each country who would like to start a business with us. Partners from each country will pay a loyalty for a given period to have the sole right for the production and sales of the product.

The brochure has a further product description and a diagram along with contact information to reach us. Detailed diagrams and videos are available through our blog and QR code.

This product is patented in Korea, and an international patent in Japan, China, EU, and USA is currently in progress.